## CoCrMo Spezial-Lot

### CE 0124

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#### Instructions for use

##### Area of application:

Non precious cobalt based hard solder alloy acc. EN ISO 9333

##### Product description:

SILADENT CoCrMo special solder is indicated to weld cobalt based alloys. It's investible with ceramic and completely without nickel, beryllium, gallium and indium ( $\leq 0,1\%$ ).

CoCrMo Special solder, 2 mm (20 g)

REF 102807

CoCrMo Special solder, 1 mm (10 g)

REF 102878

##### Working instruction:

Objects which have to be soldered must be clean and free of all oxides. Then cover the area to be soldered with flux. Mix the flux with water to a pasty consistence. Finally heat up the area slowly with a torch and deposit the solder evenly within the soldering gap.

##### Composition (in % by mass):

Co: 62.0 Cr: 29.0 Mo: 4.0 Si: 3.0

Other constituents: C - Mn - Fe

##### Melting range (°C):

Solidus-/liquidus temperature: 1.071 - 1.260 °C

Flow temperature: 1.320 °C

##### Safety hint:

Metal dust is harmful to your health. When deflasking and blasting use a suction extraction system and breathing mask.

##### Contraindications, side effects and interactions with other dental alloys:

In cases of hypersensitivity (allergy) to the constituents of the alloy. Hypersensitivity reactions (allergies) or electrochemically-induced local dysaesthesia have been reported in individual cases. A Patch-Test is thus advisable. Galvanic effects can occur under proximal or antagonistic contact with dentures of different alloys. The alloy can affect MRI results. Each supply is identified by a lot-number. Take note of this number on the patient's file in order to trace down the product.

##### Storage conditions:

No special storage conditions are required.

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