Gipsmodellglänzer

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Gypsum model gloss

For polishing gypsum models

An environment-friendly waterproofing gloss for gypsum models with the effect of keeping the model free from water, fat and dust, makes a shiny surface.

Working instruction:

The ready polished gypsum model must be clean and dry. Place the model directly in a separate basin filled with gypsum model gloss which is a ready-to-use solution. The model has to be covered with the liquid completely and stays therein for about 30 minutes. After taking the model out, it has to be dried and polished with a soft fabric. The model gloss can be used several times. Replace the solution when the liquid becomes "cloudy".

Application:

The gypsum model gloss can be used for different applications, e. g. for study models, orthodontic models, checkbite casts, showing and control models.

Contraindication

The model's surface will be sealed and it may occur that hardener and insulating agents do not adhere very well. In combination with the production of acrylic dentures it may happen that the colour of the acrylic will suffer.

Gypsum Model gloss is environmentally friendly

There is no danger for the environment and the wastewater.

Packing size:

1 x REF 603151 4.500 ml canister