# Silaform® 80 mittelhart 1:1

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# Silaform® 80 medium hard

## Instructions for use

**Type: Kneadable two-component, addition-curing silicone.**

### Technicle data:

Mixing ratio A : B	1:1
Working time at 23 °C	1' 15"
Curing time at 23 °C	3'
Shore A hardness after 24 hours	> 80
Shrinkage	< 0,01 %
Storage	36 months in original sealed container

### Application:

Measure equal amounts of components A and B using the differently marked measuring scoops. Thoroughly knead the two parts together for approx. 30 seconds until the silicone is all one colour. The impression can then be taken.

### Important:

To prevent unwanted reactions, such as premature curing, never measure out the two components using the same hand or the same measuring scoop. Close the container again carefully after each use.

### Dangerous reactions:

No known dangerous reactions, if stored and used properly.

### Toxicity:

None known

### Directions:

Store containers tightly sealed in a well-ventilated place. Keep containers dry. Keep away from water, alkalis, strong acids and oxidising agents.

### Ecological information:

Present experience suggests that there is no detrimental effect on the environment, if it is used properly and current regulations are observed. It is non-biodegradable and insoluble in water when cured. It can be easily separated from water by filtering.

### Please note:

Cured silicone is not harmful to the environment and can be disposed of along with domestic waste, as long as this does not contravene local bye-laws.

Technical recommendations are based on tests and findings from work in our development laboratory and can only be regarded as guidelines. SILADENT products are subjected to strict quality controls. We reserve the right to make technical changes. Further information can be obtained from SILADENT technical services in Goslar, Germany (+49 (0) 5321/37790) or from our sales representatives.

## Further questions:

Contact SILADENT technical services  
(Tel.: +49 (0) 53 21 – 37 79 26)  
or our sales representatives.

Date of information:  
02/2018