Silaform® 85 K

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Silaform® 85 K

Instructions for use

Type: Kneadable two-component , condensation-curing silicone with paste hardener.

Technicle data:

Mixing ratio A : B	100 : 3
Working time at 23 °C	1' 15"
Curing time at 23 °C	3'
Shore A hardness after 60 minutes	> 80
Shore A hardness after 24 hours	> 85
Shrinkage	< 1 %
Storage	36 months in original sealed container

Application:

Knead the Silaform® 85 K and Silaform® paste hardener thoroughly (1 measuring scoop of silicone to 4 cm paste hardener) until the silicone is all one colour. The impression can then be taken. Exact reproduction of detail is achieved due to the smooth initial consistency. When cured, Silaform® 85 K has excellent mechanical properties, can be trimmed easily and is resistant to inorganic chemicals.

Dangerous reactions:

No known dangerous reactions, if stored and used properly.

Toxicity:

None known

Directions:

Store containers tightly sealed in a well-ventilated place. Keep containers dry. Keep away from water, alkalis, strong acids and oxidising agents.

Ecological information:

Present experience suggests that there is no detrimental effect on the environment, if it is used properly and current regulations are observed. It is non-biodegradable and insoluble in water when cured. It can be easily separated from water by filtering.

Please note:

Cured silicone is not harmful to the environment and can be disposed of along with domestic waste, as long as this does not contravene local bye-laws.

Technical recommendations are based on tests and findings from work in our development laboratory and can only be regarded as guidelines. SILADENT products are subjected to strict quality controls. We reserve the right to make technical changes. Further information can be obtained from SILADENT technical services in Goslar, Germany (+49 (0) 5321/37790) or from our sales representatives.

Further questions: Contact SILADENT technical services
(Tel.: +49 (0) 53 21 – 37 79 26)
or our sales representatives.

Date of information:
02/2018