Silaform® Gingiva Sep

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Silaform® Gingiva Sep

Instructions for use

Silaform® Gingiva Sep is a separating agent spray for use with A-silicones consisting of a solution of paraffins in aliphatic hydrocarbons. Silaform® Gingiva Sep is intended for extraoral application only.

Silaform® Gingiva Sep prevents different A-silicone impression materials bonding together, ensuring optimal separation of the two different impression materials after curing.

Application when fabricating gingival masks:

Direct fabrication technique, application in the impression: Place a wax border around the area of gingiva to be reproduced. Then spray this area with Silaform® Gingiva Sep (REF 127301) to ensure separation. Fabricate the gingival mask in the usual way.

Indirect fabrication technique on the model: Spray the inside of the impression or overcast with Silaform® Gingiva Sep to ensure separation. Then fabricate the gingival mask in the usual way.

Important information:

Do not store above 25°C. Do not use after the expiry date.

Hazardous reactions:

Protect against heat and direct sunlight. Forms explosive gas mixtures with air. Risk of container explosion if exposed to high temperatures.

Toxicology:

Irritates the skin and mucosa. No known sensitization effect. May cause slight irritation to the eyes. Our findings indicate that the product can be used without any risks to health, provided it is handled correctly and in accordance with standard occupational hygiene measures.

Ecological information:

The product contains environmentally hazardous substances and must not be discharged into the groundwater.

Please note:

Can be incinerated in a suitable facility in accordance with local regulations. Waste code number: 14 06 03 other solvents and solvent mixtures.

Our recommendations on technique are based on tests and findings from our development laboratory and should only be regarded as guidelines. SILADENT products are subjected to strict quality controls. We reserve the right to make technical changes. For further questions please contact SILADENT technical services (Tel.: +49 (0) 53 21 – 37 79 25 / 26) or our sales representatives.

Further questions: Contact SILADENT technical services
(Tel.: +49 (0) 53 21 – 37 79 26)
or our sales representatives.

Date of information:
02/2018