GipEx Tabs

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GipEx Tabs

GipEx Tabs High reactive binder for dental gypsum and phosphate bound investments in the gypsum separator. Prevents waste pipe blockage, reduces unpleasant smells and facilitates separator cleaning.

Residues and blockages in waste pipes are hard to stop, even with regular cleaning of the gypsum separator. By trimming 15 to 40 percent of gypsum waste is caused and will not be held back by usual collecting tanks; Sooner or later the result is pipe blockage.

One pill per week is enough and only clear water will run out the pipes.

GipEx Tabs are environmentally friendly: No matter how strict your local regulations for the amount of gypsum per liter are allowed to escape into the outlet; GipEx Tabs only lets a few milligrams per liter through and is guaranteed to be way below your legal limit. GipEx Tabs are non-toxic and fulfill all European Standards. GipEx Tabs are safe to use, neutral, non-harmful substances and they have been developed according to the current anti-pollution specifications.

GipEx Tabs simplify cleaning: See for yourself, it couldn't be easier to empty the separator. GipEx Tabs prevent flag-formation and hardening. No hardening? No need to scrape out the chamber!

Hygienic and odourless: Clean water doesn't smell and particulate matter are bounded. Other liquids and detergents do not belong in the gypsum separator.

Instructions for use: After having cleansed your separator, simply put one GipEx Tab in the provided cage, and hang both in the gypsum separator; so that the tablet is totally immersed, but not in the direct waste water flow. The ideal position is in the second overflow chamber. After one hour GipEx Tabs starts to work as a flocculent. The activated agents separate even the finest suspended gypsum particles, which sink to the bottom of the separator and that avoids unpleasant smells. Only clean water escapes.

Packing (supplied with one cage): 2 x pieces REF 207410; 10 x pieces REF 207411; 25 x pieces REF 207412

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