**Instruction Manual****Intended use:**

Poly BioStar ML are tooth-coloured multilayer milling blanks made of PMMA for the manufacture of dental structures such as crowns and bridges or implant-supported temporary restorations using milling CAD/CAM technology. Colours: A1, A2, A3, A3.5, B3 & C2 (Other colours on demand)

Indications:

- Temporary anterior and posterior crowns
 - Temporary anterior and posterior bridges with up to two adjacent pontics
 - Implant supported temporary restorations
- Maximum recommended usage without check-up by dentist: 12 months

Contraindications:

- Use for dental permanent restorations
- Bridge designs involving more than two adjacent pontics
- Patients suffering from parafunctional habits such as bruxism
- Restorations should not be made with Poly BioStar ML in case of a known allergic reaction to any of its chemical constituents.

Design specifications:

Handling this medical device should only be allowed to trained staff. Dental structures are designed in CAD software. A 5-Axis milling machine is required, to reach undercut areas. When designing the objects, the minimum wall thickness should not be less than 0.9 mm in the cervical area and 1.5 mm in the occlusal area. The connection cross sections between crowns and pontics of a construction must not be less than 12 mm². In general all constructions should be designed in a way to enable the compensation and the transfer of the mastication forces to the abutment teeth. A maximum of one unsupported end unit (cantilever unit) is permitted. A Maximum of 6 units in multi-unit bridges for temporary fixed restorations is permitted. In posterior region, no more than one pontic may be positioned between two abutment crowns.

Milling instructions

When positioning the blanks in the milling machine, pay attention to the correct colour gradient. The incisal layer is marked on the blank. For the design and positioning of the construction in the blank the individual layer heights can be taken from the following nesting table:

Further processing: Tungsten carbide burs like the one you use for soft reline materials are suitable for rough adaption. For the finalization common cross-toothed milling tools can be used.

Finishing and polishing: The structure should be finished and polished in the usual manner for PMMA dental material.

Veneering: After milling, Poly BioStar ML can be faced using conventional veneering materials. MMA based materials are preferred. When veneering composite, Poly BioStar ML must be bonded following the corresponding composite's instructions for use.

Cleaning: In ultrasonic bath (max. 40 °C) with mild cleaning agent

Avoid excessive brushing which could cause abrasive wear and do not to immerse it in alkalis, acids, acetone or other substances than can damage acrylic materials

PRECAUTIONS

- Consider the physical properties of the material to use.
- Dental bridges in posterior region: Not more than two interdental connectors in acrylic resin.
- During polishing avoid breathing in dust.
- Storage conditions: No special measures required.

Disposal: Residues from processing blanks have to be disposed of as residual waste.

Side effects: In individual case, rash, sensitivity or allergies may occur by using the PMMA dental prosthesis, please stop using and consult the physician.

Interactions: Potential interactions with other products used in the treatment are very unlikely because the product is a totally polymerized material based on PMMA. In order to ensure that the selected restoration can be placed safely and durably, the dentist should consider known interactions and cross reactions of the product with other materials already present in the oral environment of each patient before using the product.

Re-use: The device can be reused as long as there is space available to mill the structures. Clean the device after each use and store it properly identified in its box. **Caution:** Reuse dentures and other used milled dental structures in other persons can infect diseases

INFORMATION FOR THE PATIENT

The patient should be informed of the following points:

- Normal oral hygiene must be maintained. Direct the patient to adopt daily hygiene protocols*, suggest him to avoid excessive brushing which could cause abrasive wear.
 - The material is not visible at x ray (except the SCAN discs).
 - Material is resistant to abrasion but heavy bites should be avoided.
- In case of an allergic reaction, seek medical advice.

***Cleaning protocol:** Patients should clean dental prosthesis at least daily. Brush them with a soft-bristled brush and nonabrasive denture cleanser to remove food, plaque and other deposits.

- The material is for dental use only.

The manufacturer is not responsible for damage caused through improper use or any other use than those explained in these instructions.

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