# SilaPrint gingiva 385 nm

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### Produktbeschreibung

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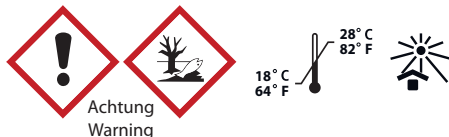
## Instructions for use

### Product description

SilaPrint gingiva is a light-curing resin for the production of gingival masks using 385 nm LED based stereolithography. For obligatory post curing of SilaPrint gingiva built parts, the high performance light curing unit Otoflash G171 is recommended, which is optimal balanced to the production process.

### Processing

- During the application of SilaPrint gingiva we recommend wearing personal protective equipment f. i. suitable gloves, safety glasses etc.
- SilaPrint gingiva should be well shaken approx. 1 hour before use.
- Pour SilaPrint gingiva carefully into the pre-determined container of the production unit.
- Remove all bubbles with a cleaned object.
- Select the product specific parameter settings for your process.
- When building process is finished a direct post treatment is recommended.
- After the platform is taken up a dripping off time of approx. 10 minutes is recommended.
- Resin coated parts get clean with isopropanol (97 %) within a time of about 10 minutes. For a particular effective cleaning with isopropanol the use of explosion-protected ultrasonic units is recommended.
- Post-curing  
Otoflash G171 with 2 x 3000 flashes under inert gas conditions (nitrogen), rotate components in between
- Impurity due to the construction or a break of the material and impurity due to operation mistakes cannot be excluded. Thanks to the low viscosity however it is possible to filtrate SilaPrint gingiva. It is recommended to regularly take out the container of the production unit, to homogenize and filtrate the content. Upcoming entrained bubbles get eliminated through a holding time of approx. 1 hour with SilaPrint gingiva.

**To avoid damage of material quality does not expose the liquid material to light under any circumstances. Variations from the described manufacturing process may lead to modified mechanical characteristics and colour variations of the SilaPrint gingiva material.**

### SILADENT Dr. Böhme & Schöps GmbH

Im Klei 26  
DE-38644 Goslar

Tel.: +49 (0) 53 21 / 37 79 - 0  
Fax: +49 (0) 53 21 / 38 96 32

info@siladent.de  
www.siladent.de

## SilaPrint gingiva 385 nm

### Instructions for use

#### Possible dangers

Please note the safety instructions for processing SilaPrint gingiva:

**Hazard statements: H315** Causes skin irritation. **H319** Causes serious eye irritation. **H317** May cause an allergic skin reaction. **H411** Toxic to aquatic life with long lasting effects.

**Precautionary statements: P261** Avoid breathing dust/fume/gas/mist/vapours/spray. **P273** Avoid release to the environment. **P280** Wear protective gloves/eye protection/face protection. **P308+P313** If exposed or concerned: Get medical advice/attention. **P333+P313** If skin irritation or rash occurs: Get medical advice/attention. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

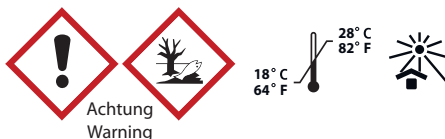
**Contains:** Urethane acrylate in HPMA, phenyl bis(2,4,6-trimethylbenzoyl)-phosphineoxide, Pigments

#### Lot number / Expiry date

The lot number and the expiry date are indicated on each SilaPrint gingiva packaging. In case of claims please always indicate the lot number of the product. Do not use the product after the expiry date.

Inhalt/Contents: 1,0 kg

 267302

Date of information: 02/2019

Printing date: 02/2019

**SILADENT Dr. Böhme & Schöps GmbH**

Im Klei 26

DE-38644 Goslar

Tel.: +49 (0) 53 21 / 37 79 - 0

Fax: +49 (0) 53 21 / 38 96 32

info@siladent.de

www.siladent.de